

HURRISEAL[®] DENTIN DESENSITIZER

Product # 0283-0697-82

UPC 723797697825

Ingredients: Hydroxyethyl Methacrylate (HEMA), Sodium Fluoride;
Inactive: Benzalkonium Chloride (preservative), water.

HurriSeal[®] Dentin Desensitizer is an aqueous solution, which contains no photo-initiator. It is a versatile dentin desensitizer capable of acting as a wetting agent when bonding strength is a consideration.

Directions for Use

Indications for Use: HurriSeal Dentin Desensitizer with HEMA, fluoride and benzalkonium chloride, is to be applied prior to restorative procedures by the dental professional.

For Dental Use Only. Not for ingestion or for rinsing the oral cavity.

Warning: Avoid contact with eyes and skin. If accidental contact occurs, flush immediately with water. Consult physician if eye contact occurs.

For All Applications: Place a small quantity of HurriSeal Dentin Desensitizer in a dappen dish. Use a micro-tip or composite brush as the applicator.

Cervical/Root Surface Application:

1. Polish the area to be treated with flour pumice and water to remove the smear layer.
2. Rinse completely.
3. Blot surface with gauze or cotton pellets and dry with indirect airflow (manage according to patient discomfort).
4. Apply the first coat of HurriSeal Dentin Desensitizer with a gentle rubbing motion (burnish). For best results, use a micro-tip or composite brush.
5. Allow to **air dry** for 20 seconds.
6. Repeat application 1 – 2 more times.
7. Gently dry with indirect airflow, testing for sensitivity (if residual sensitivity is detected, another coat may be applied).

Special Considerations:

Caution: Although HurriSeal Dentin Desensitizer is soft tissue friendly, isolation of the gingiva with gauze or cotton rolls is recommended as excess solution may cause minor tissue irritation. Post Scaling and Root Planing, allow soft-tissue healing prior to HurriSeal Dentin Desensitizer application.

PLEASE NOTE: If a Class V lesion is being restored after cervical desensitization, employ HurriSeal Dentin Desensitizer application directions for under a composite restoration. Etch may be applied if local anesthesia is used and increased bonding strength is desired.

Restorative Dental Procedures:

NON-BONDED RESTORATIONS

Conventional Alloys (Amalgam)

1. Apply HurriSeal[®] Dentin Desensitizer to all exposed dentin surfaces within the preparation. For best results, use a micro-tip or composite brush.
2. Dry with air syringe.
3. Repeat application 1 – 2 times.
4. Apply base appropriately (if necessary).
5. Proceed with restoration.

Temporary Restoration (for external fabrication restorative procedures)

1. Pumice the preparation.
2. Rinse and dry with air syringe.
3. Apply 2 – 3 coats of HurriSeal Dentin Desensitizer and **thoroughly** dry with air syringe.
4. Cement temporary restoration.

Caution: Do not use eugenol-containing products in combination with HurriSeal Dentin Desensitizer.
Eugenol neutralizes the effectiveness of the HEMA.

BONDED RESTORATIONS

HurriSeal Dentin Desensitizer is always applied after etching and before priming or bonding, when using bonding materials.

Caution: HurriSeal Dentin Desensitizer is not recommended for use in conjunction with self-etching bonding systems.

Composite Restoration and Bonded Alloys

1. Etch the preparation per directions.
2. Rinse and remove excess water by drying (**Do not desiccate the prep**).
3. Apply up to 3 coats of HurriSeal Dentin Desensitizer allowing each coat to **air dry** for 20 seconds.
4. Use the high-speed evacuator tip to remove any 'puddling' or excess solution in prep. Do not dry with air syringe.

For Best Results: Prep should still be moist. Allow HurriSeal Dentin Desensitizer to act as the wetting agent.

Crown and Bridge

1. Glass ionomer cements: Apply HurriSeal Dentin Desensitizer prior to placement of glass ionomer. Leave moist.
2. Conventional cements: Apply HurriSeal Dentin Desensitizer to non-etched tooth and dry.
3. Resin cements: Apply HurriSeal Dentin Desensitizer as recommended for bonded restoration, and then cement crown.

Storage Conditions: Do not store above 77° F (25° C). Refrigerate if not used for long periods of time. Do not freeze.

For additional information or to order Beutlich® LP products online, visit our website at: www.beutlich.com
HurriSeal is a Registered Trademark of Beutlich LP, Pharmaceuticals